



User Guide Addendum Smart Tip (1x55mm Round)

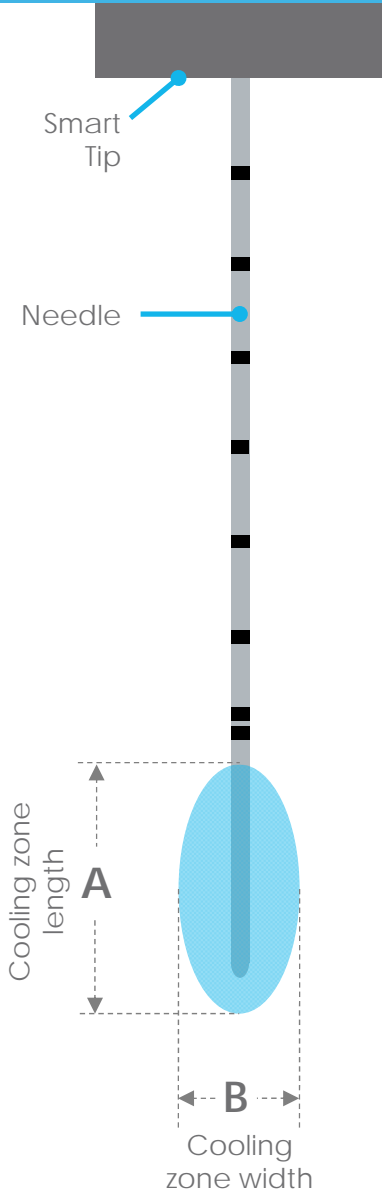

	mm (in)	
	A	B
STT0413	9.4 (0.37)	5.4 (0.21)

Dimensions are based on 40-second cooling cycle performed in agarose gel.

This document is provided as a supplement to the User Guide. Always refer to the User Guide for complete operating instructions, warnings and precautions.



WARNING! To reduce likelihood of skin injury, insert needle at least 15mm, denoted by last double marking ■.



CAUTION! Small bend radii may affect performance.

- Do not kink or permanently deform needle.
- 35mm minimum radius.

Procedure:

- Prep cycle not required for this tip type.
- Pre-pierce the skin before inserting Smart Tip.
- Insert needle to maximum insertion desired, then perform cooling cycle.
- Needle markings are at 6mm increments.