


User Guide Addendum

Smart Tip (1x90mm Sharp) with Nerve Stim

iovera^o

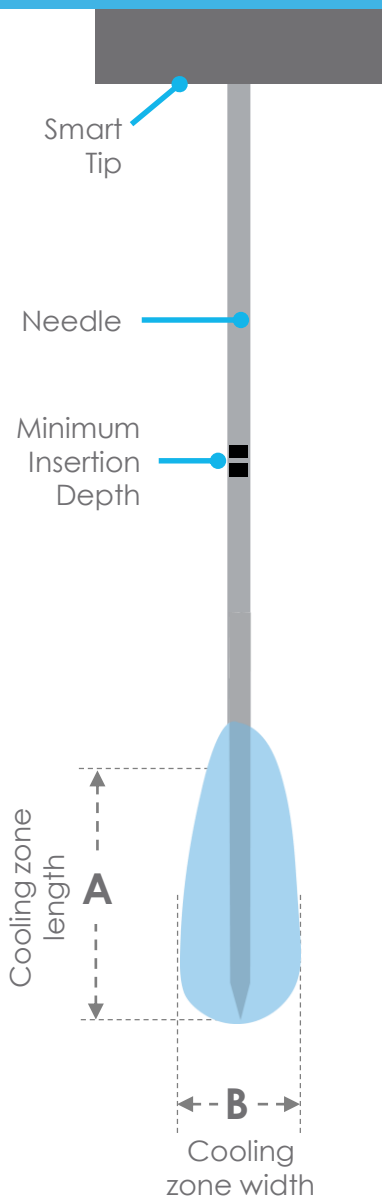

h	mm (in)	
	A	B
STT0811	16.0 (0.63)	7.1 (0.28)

Dimensions are for information purposes only. They are based on 60-second cooling cycle performed in agarose gel. The ice ball size could vary based on anatomy/physiology of patients and should not affect the efficacy of treatment.

This document is provided as a supplement to the User Guide. Always refer to the User Guide for complete operating instructions, warnings and precautions.



WARNING! To reduce likelihood of skin injury, insert needle at least 30mm, denoted by the double marking (labeled minimum insertion depth on the image) ■■.



CAUTION! Small bend radii may affect performance.

- Do not kink or permanently deform needle.
- 55mm minimum radius.

Procedure:

- Prep cycle not required for this tip type.
- Pre-piercing of the skin may make Smart Tip insertion easier.
- Insert needle to maximum insertion desired, then perform cooling cycle.

Optional Nerve Stimulation

- This iovera^o Smart Tip is enabled to be connected to a separate off-the-shelf nerve stimulator via the port on the side of the Smart Tip using the cable provided.
- Please refer to manufacturer-specific instructions for that device and observe all warnings, cautions, and precautions should the user choose to use this optional feature.