


# User Guide Addendum Smart Tip (3x8.5mm Sharp, 27 Gauge)

iovera<sup>o</sup>

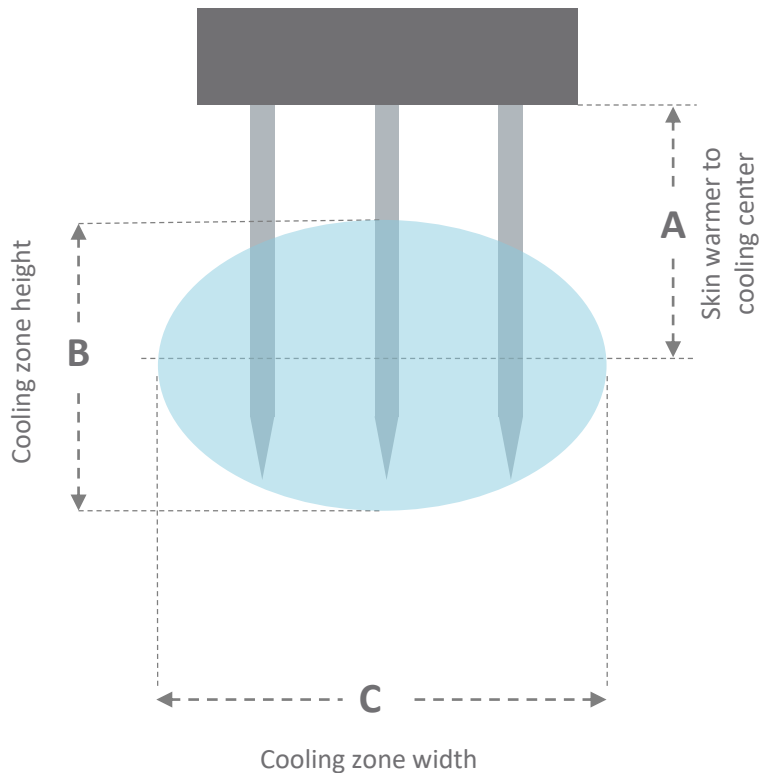

h	A mm (in)	B mm (in)	C mm (in)
STT0309	6.3 (0.25)	6.5 (0.26)	8.0 (0.31)

Dimensions are for information purposes only. They are based on 33-second cooling cycle performed in agarose gel. The ice ball size could vary based on anatomy/physiology of patients and should not affect the efficacy of treatment.

This document is provided as a supplement to the User Guide. Always refer to the User Guide for complete operating instructions, warnings and precautions.

### Procedure:

- Prep cycle **not required** for this tip type.
- Insert needle to maximum insertion desired, then perform cooling cycle.



**WARNING!** To reduce likelihood of skin injury, skin warmer must be pressed against the skin when the needle is inserted before running a treatment cycle.



**WARNING!** Needle may contact bone in low BMI patients. Patient anatomy should be assessed prior to tip selection.