


# User Guide Addendum Smart Tip (1x90mm Sharp)

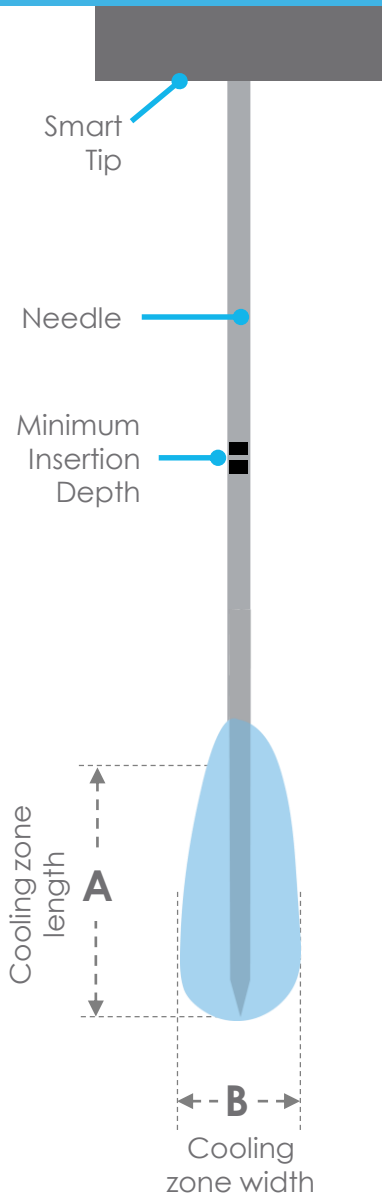

h	mm (in)	
	A	B
STT0801	16.0 (0.63)	7.1 (0.28)

Dimensions are for information purposes only. They are based on 60-second cooling cycle performed in agarose gel. The ice ball size could vary based on anatomy/physiology of patients and should not affect the efficacy of treatment.

This document is provided as a supplement to the User Guide. Always refer to the User Guide for complete operating instructions, warnings and precautions.

**!** **WARNING!** To reduce likelihood of skin injury, insert needle at least 30mm, denoted by the double marking (labeled minimum insertion depth on the image) ■.

**!** **CAUTION!** Small bend radii may affect performance.

- Do not kink or permanently deform needle.
- 55mm minimum radius.

### Procedure:

- Prep cycle not required for this tip type.
- Pre-piercing of the skin may make Smart Tip insertion easier.
- Insert needle to maximum insertion desired, then perform cooling cycle.